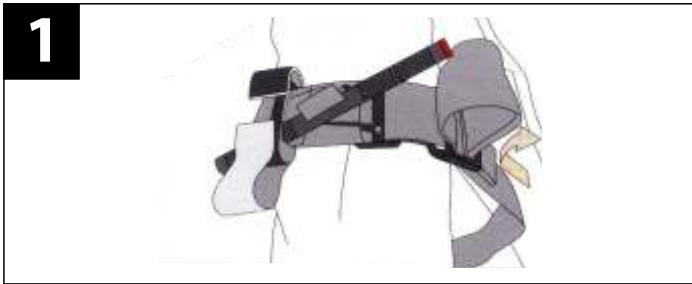
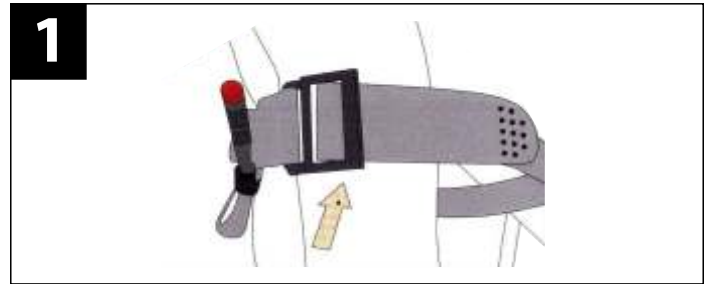
# TOURNIQUET

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




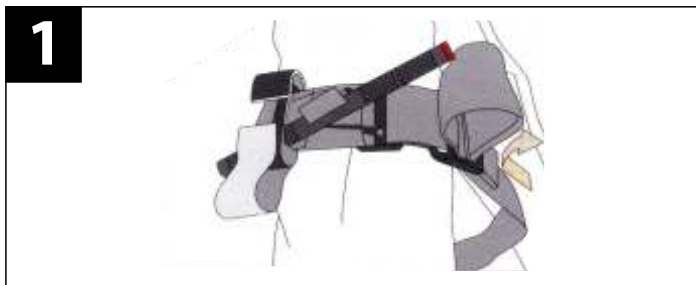
## TWO-HANDED APPLICATION

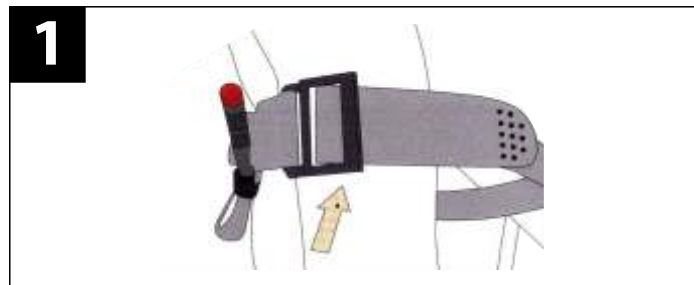Route and band around the limb, and pass the tip through the gaps for the buckle. Position the tourniquet 2-3" above the wound site.

## ONE-HANDED APPLICATION

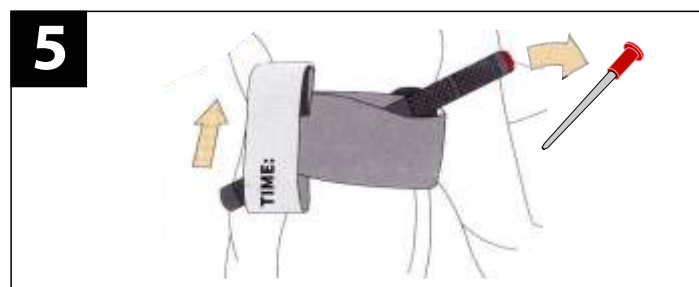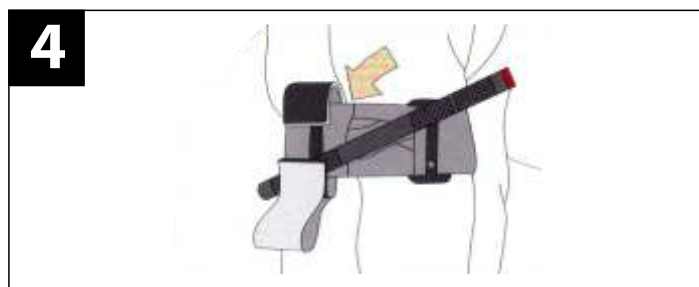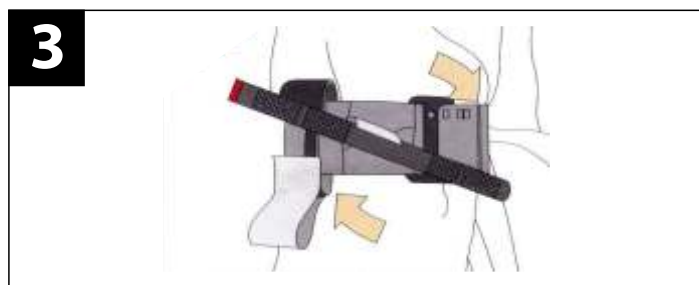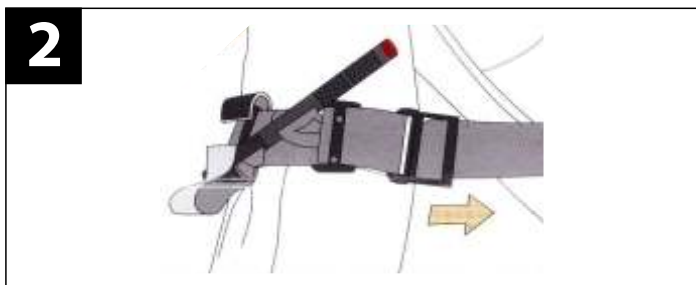Insert the injured limb through the loop in the band and position the tourniquet 2-3" above the bleeding site directly to the skin.

The band should always travel around both, the rolling pin and sliding teeth in the buckle. Pull the band tight and fasten it back on itself, all the way around the limb but not over the rod clips. The band should be tight enough so that 3 fingers cannot slide beneath it. If they can, re-tighten and secure.

Twist the rod until bleeding has stopped. Also check for distal pulse beneath the tourniquet near the wound, this should disappear.

Secure the rod inside the clips to lock it in place. Check for bleeding and distal pulse. If bleeding continues or distal pulse is present below the tourniquet, consider additional tightening or applying a second tourniquet above the first. Reassess.

Route the band between the clips and over the rod. Secure the rod and band with the TIME strap. Record time of application with the pen at the top. Consider further securing with medical tape. Ensure tourniquet is visible for medical personnel.

\*If you are unable to identify the exact site of bleeding or if you do not have sufficient time to do so, the Tourniquet can be applied effectively as proximal as possible to the limb, including on the clothing, depending on the situation. However, the Tourniquet must never be applied over hard objects in the clothing. The limb should then be re-examined as soon as possible and the Tourniquet applied directly to the skin according to the instructions.